Once inserted, check the following to see if it is the right size:

- It does not fall out when moving or walking (if it does, try the next larger size);
- You do not feel it (if there is pressure or discomfort, try the next smaller size);
- When you cough, sneeze, laugh, or exercise, your bladder does not leak (or leaking is significantly reduced);
- You can urinate (empty your bladder) with your uresta® in place – the product should not stop you from urinating or having a bowel movement when properly fitted.

Removal Steps

1. Wash your hands with a mild soap and warm water. Rinse thoroughly.
2. Separate the labia (the opening or outer lips) of the vagina with one hand. With the other hand, insert index finger to locate the handle of the uresta®.
3. Grasp the handle with your index finger and thumb, and pull the uresta® down and out of the vagina.

If you experience any difficulty in reaching or removing the product, try:

- A few deep breaths to relax and release tension in the pelvic floor muscles;
- Give a small twist of the handle to "break the seal";
- Bear down as if trying to have a bowel movement to help move the uresta® down to a position where you can more easily grasp the handle.

Product Care

Always wash your hands with mild soap and warm water, rinsing thoroughly, before inserting or removing uresta®.

You should remove uresta® daily to clean it. To clean:

1. Use warm water and a mild, non-perfumed soap to wash your uresta®. Rinse thoroughly with water after washing;
2. Dry with a towel or cloth;
3. Place in bottom unit of uresta® case and leave the case open to air dry.

Replace your uresta® if a defect (e.g. a crack) develops and replace your uresta® once a year to ensure continued proper fit, comfort, and safety.

Troubleshooting

If you have difficulty or you are experiencing discomfort inserting:

- Use a water based lubricant
- Take deep breaths to relax pelvic muscles
- Try inserting the next smaller size
- If problems persist, consult a physician

If the product does not control or significantly reduce your bladder leakage or will not stay in place:

- Try one size larger
- If the product falls out only during bowel movements, remove it before a bowel movement, as per instructions and replace it after, as per instructions
- If problems persist, consult a physician

If the product is uncomfortable or if you cannot empty your bladder:

- Try one size smaller
- If problems persist, consult physician

If you experience difficulty removing:

- Take deep breaths to relax pelvic muscles and gently bear down as if trying to have a bowel movement
- If unable to remove, consult a physician within 24 hours

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