## Instructions for Use

**Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.**

### Introduction

Thank you for choosing the uresta® bladder support for control or reduction of stress urinary incontinence. These instructions will outline what you need to know about using this product, including insertion and removal, and care of your uresta® bladder support.

uresta® may be used in different ways to suit your lifestyle. It can be inserted in the morning and removed at bedtime everyday or used just for specific activities such as exercise and removed after the activity. Remove uresta® before sexual intercourse and during menstruation.

### How the uresta® bladder support works

uresta® is made from a thermoplastic elastomer, is not made with natural rubber latex, and has been designed and tested for biocompatibility and use in the vagina. When properly inserted into the vagina, uresta® provides additional support through the vaginal canal to reduce or control involuntary urine loss due to stress urinary incontinence.

### Indications for Use

The uresta® bladder support is indicated for use in adult women over 18 years of age who experience stress urinary incontinence (involuntary urine loss, such as a spurt of urine that occurs when you cough, laugh, sneeze, or with some physical tasks).

### Contraindications – do not use this product:

- If you are under 18 years of age
- If you are pregnant
- During sexual intercourse or menstruation
- If you have had recent vaginal surgery

### Do not use the product if you have any of the following:

- Abnormal vaginal bleeding or excessive vaginal discharge and/or foul odour associated with discharge
- Vaginal soreness or itching

### Precautions

- If you suffer from pre-existing conditions, including previous vaginal or related surgery, atrophy, or prolapse, please consult a physician prior to using your uresta® bladder support.
- uresta® is not indicated for use in patients experiencing vaginal prolapse.
- Please refer to the Product Care Section of this Instruction for Use (IFU) for cleaning instructions of the product. Improper washing and care of the product may lead to a risk of infection.
- Replace your uresta® product as directed in the Product Care Section of this IFU. Failure to do so may lead to improper fit, discomfort, or infection.

### Do not use this product if:

- The tamperproof seal on the retail package is broken at time of purchase
- Inspection of the uresta® bladder support reveals cracks or other defects

Your uresta® is for personal use only. For health and safety reasons, do not share it with any other person.

### Warning

Stop using uresta® and consult a physician if any of the following occurs:

- Difficulty urinating
- Vaginal itching, pain or discomfort with the use of the product
- Unexpected vaginal bleeding
- Change in colour, consistency, odour, or amount of vaginal discharge
- Burning with urination and/or the need to urinate more often

### Adverse Effects

Clinical studies of uresta® have not shown any significant complications. No association has been established between uresta® and toxic shock.

Toxic Shock Syndrome (TSS) is considered a serious and sometimes fatal disease caused by toxin-producing strains of Staphylococcus aureus bacterium. TSS has been linked to high absorbency and prolonged use of tampons. Symptoms of TSS often mimic the flu and can include sudden high fever, vomiting, diarrhea, dizziness, fainting or a rash that looks like sunburn during your period or a few days after. If you experience any of these symptoms while using the product, remove and immediately contact a physician.

### Insertion Steps

1. Remove the uresta® from the box and packaging material; note that the size of each bladder support is identified by a number on one side of the handle.
2. Wash your hands with a mild soap and warm water. Rinse thoroughly.
3. The use of a water-based lubricant designed for vaginal use (available at a local pharmacy) is recommended to be used to assist with inserting the uresta® bladder support. It is usually helpful to place one foot on a stool or toilet seat cover to prepare for insertion of the uresta® bladder support.
4. If using the product for the first time, select the size 4 uresta® and grasp the handle with one hand. With the other hand, place a small amount of lubricant on the tip of the tapered end (see Figure 1).
5. Holding the uresta® in one hand, use the other hand to separate the labia (the opening or outer lips) of the vagina and direct the tip of the tapered end into the vagina (see Figure 2).
6. Gently push the uresta® product into the vagina aiming towards the spine. (see Figure 3 and 4)

As the product enters the vagina the tapered end will help to open the vagina. You will feel some pressure and resistance at the opening as you push the widest part of the uresta® in and below the pubic bone, at which point the pressure and resistance will subside.

Once inserted, the end of the handle should be approximately 2.0 cm or approximately ¾ of an inch inside the opening to the vagina and you should be able to easily touch the end of the handle and grip the handle for removal. You should not feel the uresta® when properly sized and inserted.

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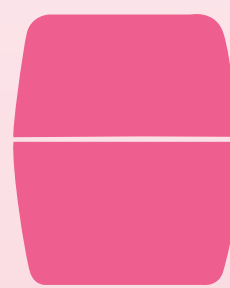
52mm|2.0"

Case

Once inserted, check the following to see if it is the right size:

- It does not fall out when moving or walking (if it does, try the next larger size);
- You do not feel it (if there is pressure or discomfort, try the next smaller size);
- When you cough, sneeze, laugh, or exercise, your bladder does not leak (or leaking is significantly reduced);
- You can urinate (empty your bladder) with your uresta® in place – the product should not stop you from urinating or having a bowel movement when properly fitted.

#### Removal Steps

1. Wash your hands with a mild soap and warm water. Rinse thoroughly.
2. Separate the labia (the opening or outer lips) of the vagina with one hand. With the other hand, insert index finger to locate the handle of the uresta®.
3. Grasp the handle with your index finger and thumb, and pull the uresta® down and out of the vagina.

If you experience any difficulty in reaching or removing the product, try:

- A few deep breaths to relax and release tension in the pelvic floor muscles;
- Give a small twist of the handle to "break the seal";
- Bear down as if trying to have a bowel movement to help move the uresta® down to a position where you can more easily grasp the handle.

#### Product Care

Always wash your hands with mild soap and warm water, rinsing thoroughly, before inserting or removing uresta®.

You should remove uresta® daily to clean it. To clean:

1. Use warm water and a mild, non-perfumed soap to wash your uresta®. Rinse thoroughly with water after washing;
2. Dry with a towel or cloth;
3. Place in bottom unit of uresta® case and leave the case open to air dry.

Replace your uresta® if a defect (e.g. a crack) develops and replace your uresta® once a year to ensure continued proper fit, comfort, and safety.

#### Troubleshooting

If you have difficulty or you are experiencing discomfort inserting:

- Use a water based lubricant
- Take deep breaths to relax pelvic muscles
- Try inserting the next smaller size
- If problems persist, consult a physician

If the product does not control or significantly reduce your bladder leakage or will not stay in place:

- Try one size larger
- If the product falls out only during bowel movements, remove it before a bowel movement, as per instructions and replace it after, as per instructions
- If problems persist, consult a physician

If the product is uncomfortable or if you cannot empty your bladder:

- Try one size smaller
- If problems persist, consult physician

If you experience difficulty removing:

- Take deep breaths to relax pelvic muscles and gently bear down as if trying to have a bowel movement
- If unable to remove, consult a physician within 24 hours

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